

Instructions

Carefully read through the following instructions before starting.


1. Identify parts of the testing kit in front of you:



Specimen tube




Cartridge

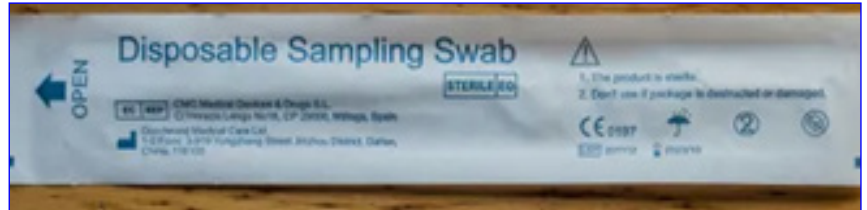

Swab


2. Blow your nose into a tissue to get rid of any excess mucus. Discard tissue in bin and sanitise your hands.

3. Open the swab package and remove the swab, taking care not to touch fabric end of swab:

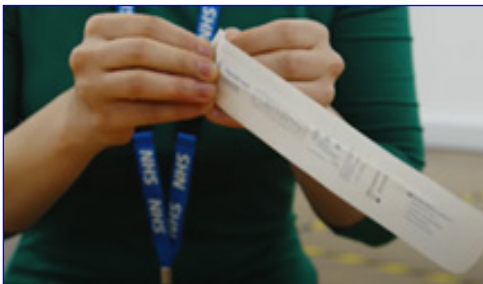


4. Gently place the swab 2.5cm into one nostril or until you feel some resistance. Roll the swab 10 times along inside of nostril. Repeat this on your other nostril with the same swab.

5. Now put the same end of the swab that you've just used into the extraction tube. Pick up tube and rotate the swab in the liquid to squeeze out as much fluid as possible:

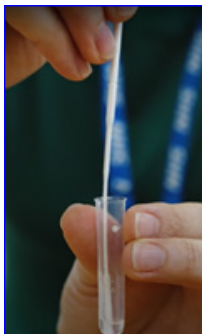


6. Remove the swab while squeezing the tube and place swab back in packet before disposing of it in the bin.

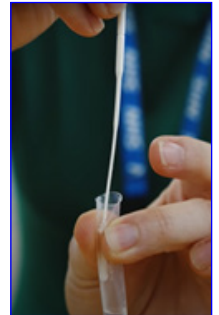

7. Press the cap onto the tube:



8. Turn the tube upside down and gently squeeze out two drops onto the specimen well of the cartridge. Place tube in bin.

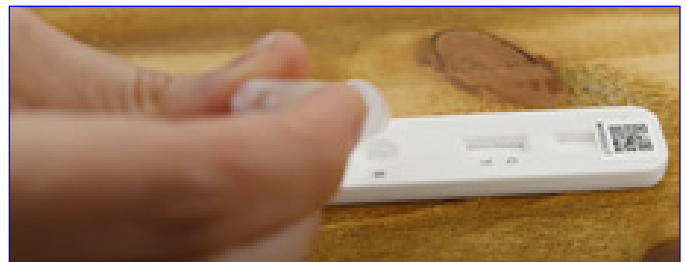

9. Start timer (30 Minutes).

10. You have finished.